


**AM-TSL-F8 (2-point)**



Item/Color	Size	SWL
AM-TSL-L-F8	46" x 86"	1000 Lb/ 454 Kg
AM-TSL-W-F8	50" x 86"	1000 Lb/ 454 Kg
AM-TSL-TS-L-F8	46" x 86"	1000 Lb/ 454 Kg
AM-TSL-TS-W-F8	50" x 86"	1000 Lb/ 454 Kg

**SLING PLACEMENT ON BED**

- PLACE BARRIER SHEET OR CHUCK BETWEEN PATIENT AND SLING - DO NOT LAY PATIENT DIRECTLY ON MESH.
- FOOT LABEL SHOULD BE AT FOOT END.
- PLACE PILLOW BETWEEN KNEES WHEN LOG ROLLING PATIENT FOR HIP PRECAUTIONS.

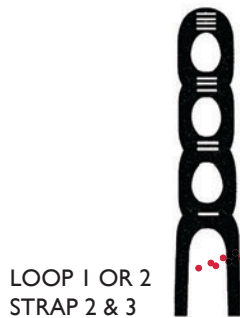


LOOP 1 OR 2  
STRAP 2 & 3



**LOG ROLLING**

- TO LOGROLL : CHOOSE TWO STRAPS, ONE CLOSE TO PATIENT SHOULDER AND ONE CLOSE TO PATIENT HIPS.
- USE SHORTEST ATTACHMENT POINT ON FIGURE 8 STRAP.



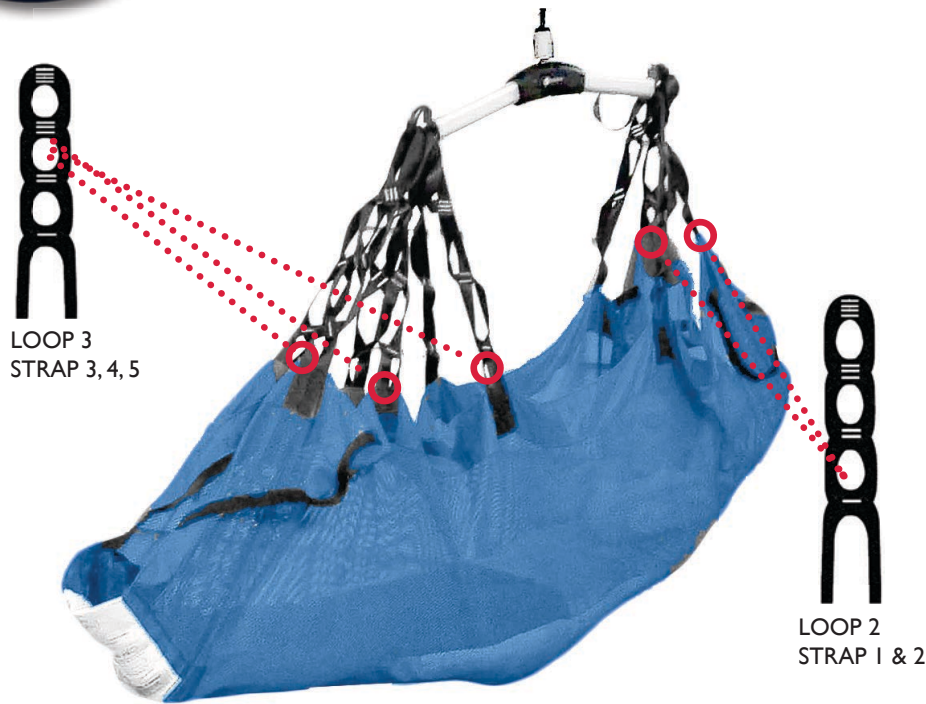

**SUPINE TRANSFER**

- ATTACH **STRAPS 1-2** ON BOTH SIDES TO THE 2 HOOKS NEAREST TO PATIENT'S HEAD.
- ATTACH **STRAPS 3, 4 & 5** ON EITHER SIDE TO THE 2 HOOKS NEAREST TO FEET.

**RECLINED TRANSFER**

- POSITION HANGER BAR PERPENDICULAR TO PATIENT'S TORSO.
- ATTACH **STRAPS 1, 2, 3 & 4** ON BOTH SIDES TO THE TWO HANGER BAR HOOKS.
- DO NOT ATTACH **STRAP 5**.

