


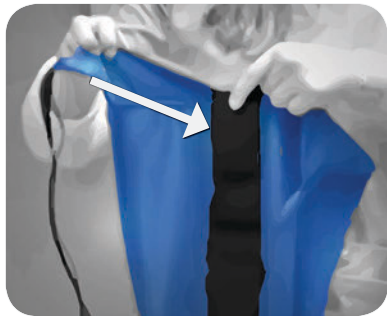

1
Before placing sling under patient, make sure the Velcro strap opens on the top.



2
Hold Velcro strap and fold one sling corner over it.



3
Wrap the other side of the sling over the top.

Use one hand to grab the middle of the wrapped sling.

Make sure strap and sling straps are tucked under the sling fabric.



4
Use the other hand to grab the folded part of the sling.

Slide folded sling under small area of patient's back.



5-6

Pull wrapped end of sling on the other side of the patient.

Unfold wrapped sling.



7-9

Unfold Velcro strap.

Attach Velcro strap securely to bed frame or other attachment point available on bed frame.



10 Attach the two sling loops to hanger bar in parallel to the body, on the opposite side.



11 Raise bed rails for safety (not shown).

Stand on the side you are turning the patient towards.



12 Use the lift to turn the patient towards you.